



ALL SUTURE ANCHOR

SOFT ANCHOR

SA

ARTHRO **VIMS**[®]

SOFT ANCHOR






ALL SUTURE ANCHOR

It is specially designed for the fixation of soft or prosthetic tissues to the bone

ARTHROVIMS SOFT ANCHOR are impacted anchor of small diameter exclusively composed of textile fibers.

The polyester body of the anchor is associated with one or two UHMWPE suture threads which are mounted on a disposable screwdriver allowing insertion into the bone.

The traction on the suture threads makes it possible to modify the shape of the body of the anchor which thus locks itself behind the cortical bone and ensures an effective anchorage.

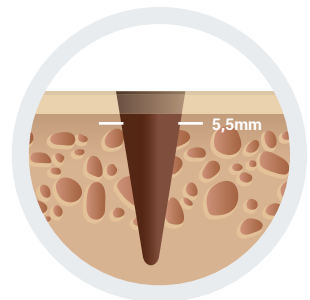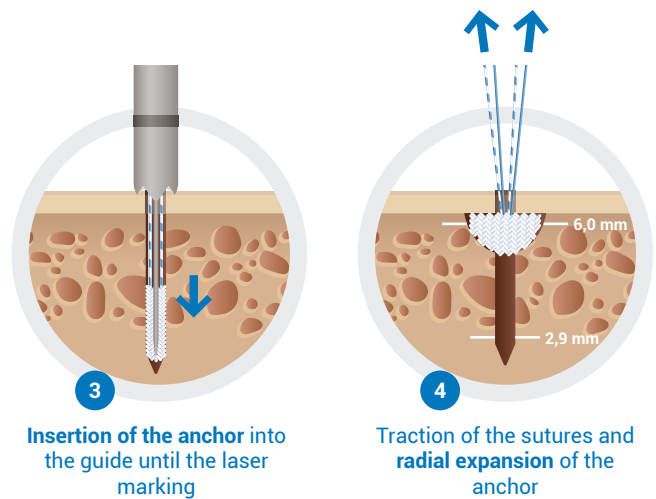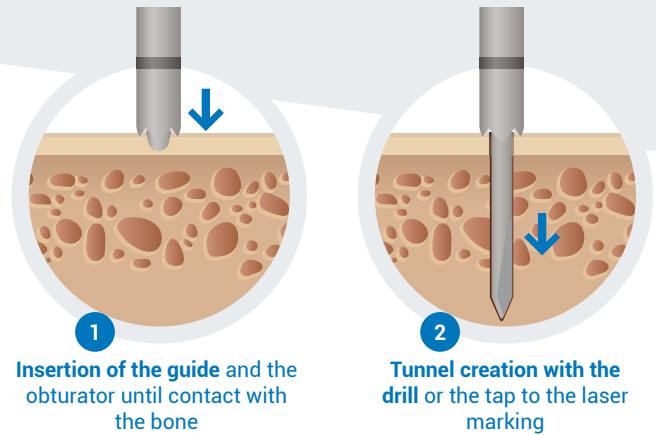

Bone economy compared to a screwed anchor

ADVANTAGES

- Bone economy
- Easy to use
- No rigid material in case of revision or in the joint in case of tearing
- Radio-transparent
- Can be used under arthroscopy or mini open (with needles)
- Biocompatible
- High pull-out resistance
- Non-absorbable

Beveled tip for better impaction in bone tissue





SHOULDER
ROTATOR CUFF,
BICEPS TENODESIS AND
REINSERTION

Ø2,9
mm



SHOULDER
SLAP, BANKART
INSTABILITY



ANKLE
TENDON AND LIGAMENT
REINSERTION

Ø1,8
mm



HAND
TENDON AND LIGAMENT
REINSERTION

Ø1,2
mm



ANKLE
TENDON AND LIGAMENT
REINSERTION

Ø1,5
mm



SUTURE ADVANTAGES

ARTHROVIMS® SOFT ANCHOR are associated with 1, 2 or 3 sutures, 1 suture and 1 tape, 1 or 2 tapes made of very high molecular weight polyethylene (UHMWPE)

HIGH TENSILE STRENGTH
EASY REALIZATION OF KNOTS
STRENGTH AND SAFETY OF SUTURES



STORAGE

ARTHROVIMS® SOFT ANCHOR must be stored in its original unopened packaging, in a clean and dry place, at ambient temperature.

PRODUCTS & INSTRUMENTATION

ARTHROVIMS® SOFT ANCHOR is packaged in a double pouch and sterilized by ethylene oxide. Instrumentation for arthroscopic procedures is available. Single use. **Do not re-sterilize.**

ARTHROVIMS® SOFT ANCHOR must be used with the corresponding instrumentation supplied by VIMS.

For any further information, please refer to the IFU.

DÉSIGNATION	TAILLE	RÉFÉRENCE	
ARTHROVIMS SOFT ANCHOR - 2 sutures : white/black&blue + needles	2,9	VIMS-029SN	
ARTHROVIMS SOFT ANCHOR - 2 tapes : blue & white/black + needles	2,9	VIMS-029TN	
ARTHROVIMS SOFT ANCHOR - 2 sutures : blue & white/black	2,9	VIMS-029S	
ARTHROVIMS SOFT ANCHOR - 1 suture : white/black - 1 tape blue	2,9	VIMS-029STB	
ARTHROVIMS SOFT ANCHOR - 1 suture : white/blue - 1 tape black	2,9	VIMS-029STG	
ARTHROVIMS SOFT ANCHOR - 2 tapes : white/black & blue	2,9	VIMS-029TB	
ARTHROVIMS SOFT ANCHOR - 2 tapes : white/blue & black	2,9	VIMS-029TG	
ARTHROVIMS SOFT ANCHOR - 3 sutures : blue - white/black & white/blue	2,9	VIMS-029S3	
ARTHROVIMS SOFT ANCHOR - 2 sutures : blue & white/blue needles rigid handle	1,8	VIMS-018SN	
ARTHROVIMS SOFT ANCHOR - 2 sutures : blue & white/black flexible handle	1,8	VIMS-018S	
ARTHROVIMS SOFT ANCHOR - 1 suture : white/black - 1 tape blue flexible handle	1,8	VIMS-018ST	
ARTHROVIMS SOFT ANCHOR - 1 suture : white/blue flexible handle	1,5	VIMS-015S	
ARTHROVIMS SOFT ANCHOR - 1 tape : white/blue flexible handle	1,5	VIMS-015T	
ARTHROVIMS SOFT ANCHOR - 1 suture : blue + needles rigid handle 9cm	1,5	VIMS-015SN	
ARTHROVIMS SOFT ANCHOR - 1 suture : white/blue + needles	1,2	VIMS-012SN	Coming soon

INSTRUMENTATION

ARTHROVIMS® SOFT ANCHOR must be used with the specific instrumentation provided by VIMS.

2265
CLASSE IIb / ANCHOR

CLASSE I / INSTRUMENTATION