

ARTHRO SIMS® SOFT ANCHOR








ALL SUTURE ANCHOR

It is specially designed for the fixation of soft or prosthetic tissues to the bone


ARTHROVIMS SOFT ANCHOR are impacted anchor of small diameter exclusively composed of textile fibers.

The polyester body of the anchor is associated with one or two UHMWPE suture threads and/ or tapes which are mounted on a disposable insertor allowing insertion into the bone.


The traction on the suture threads makes it possible to modify the shape of the body of the anchor which thus locks itself behind the cortical bone and ensures an effective anchorage.

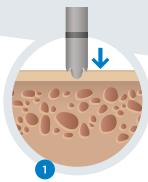


Insertion of the guide and the obturator until contact with the bone

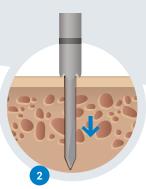


Tunnel creation with the drill or the tap to the laser marking

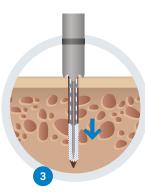

Insertion of the anchor into the guide until the laser marking

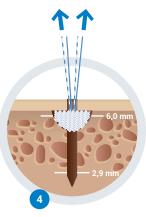

Traction of the sutures and **expansion** of the anchor

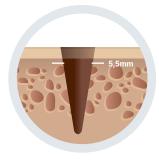

Bone economy compared to a screwed anchor

ADVANTAGES

- Bone economy
- Easy to use
- Radio-transparent
- Can be used under arthroscopy or mini open (with needles)
- Biocompatible
- High pull-out resistance
- Non-absorbable

Beveled tip for better impaction in bone tissue

Laser marking for insertion visualization

































SUTURE ADVANTAGES

ARTHROVIMS® SOFT ANCHOR are associated with 1, 2 or 3 sutures, 1 suture and 1 tape, 1 or 2 tapes made of very high molecular weight polyethylene (UHMWPE)

HIGH TENSILE STRENGTH EASY REALIZATION OF KNOTS STRENGTH AND SAFETY OF SUTURES



STORAGE

ARTHROVIMS® SOFT ANCHOR must be stored in its original unopened packaging, in a clean and dry place, at ambient temperature.


PRODUCTS & INSTRUMENTATION

ARTHROVIMS® SOFT ANCHOR is packaged in a double pouch and sterilized by ethylene oxide. Single use. Do not re-sterilize. Instrumentation for arthroscopic or mini-open procedures is available. For any further information, please refer to the IFU and/or contact us.

DESIGNATION	SIZE	REFERENCE
ARTHROVIMS SOFT ANCHOR - 2 sutures : white/black&blue + needles	2,9	VIMS-029SN
ARTHROVIMS SOFT ANCHOR - 2 tapes : blue & white/black + needles	2,9	VIMS-029TN
ARTHROVIMS SOFT ANCHOR - 2 sutures : blue & white/black	2,9	VIMS-029S
ARTHROVIMS SOFT ANCHOR - 1 suture : white/black - 1 tape blue	2,9	VIMS-029STB
ARTHROVIMS SOFT ANCHOR - 1 suture : white/blue - 1 tape black	2,9	VIMS-029STG
ARTHROVIMS SOFT ANCHOR - 2 tapes : white/black & blue	2,9	VIMS-029TB
ARTHROVIMS SOFT ANCHOR - 2 tapes : white/blue & black	2,9	VIMS-029TG
ARTHROVIMS SOFT ANCHOR - 3 sutures : blue - white/black & white/blue	2,9	VIMS-029S3
ARTHROVIMS SOFT ANCHOR - 2 sutures : blue & white/blue needles rigid handle	1,8	VIMS-018SN
ARTHROVIMS SOFT ANCHOR - 2 sutures : blue & white/black flexible handle	1,8	VIMS-018S
ARTHROVIMS SOFT ANCHOR - 1 suture : white/black - 1 tape blue flexible handle	1,8	VIMS-018ST
ARTHROVIMS SOFT ANCHOR - 1 suture : white/blue flexible handle	1,5	VIMS-015S
ARTHROVIMS SOFT ANCHOR - 1 tape : white/blue flexible handle	1,5	VIMS-015T
ARTHROVIMS SOFT ANCHOR - 1 suture : blue + needles rigid handle 9cm	1,5	VIMS-015SN

INSTRUMENTATION

ARTHROVIMS® SOFT ANCHOR must be used with the specific instrumentation provided by VIMS.

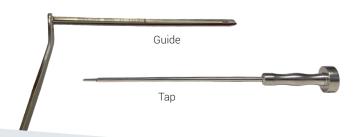







VIMS SA 10 AVENUE DE FONTRÉAL - EUROCENTRE 31620 VILLENEUVE LES BOULOC - FRANCE TÉL. +33 5 34 45 09 09 - FAX. +33 5 61 22 85 84

VIMS-SYSTEM.COM



CLASSE I / INSTRUMENTATION

Non-contractual pictures